
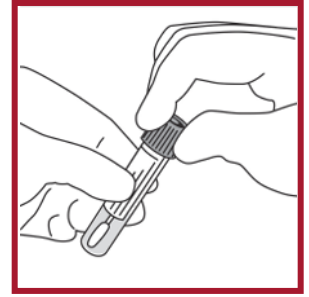
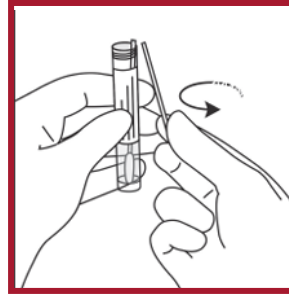
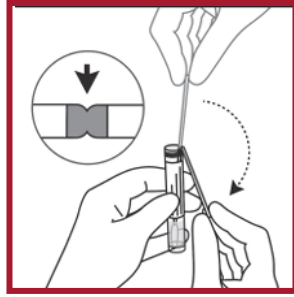


ESwab Collection Instructions


READ ALL INSTRUCTIONS BEFORE COLLECTION

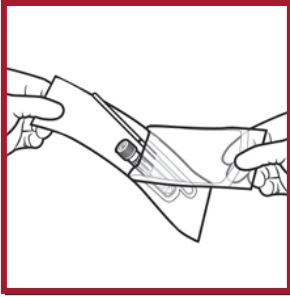

Wash hands thoroughly before starting.

1. Open the ESwab sample collection pouch and remove the tube and swab.
2. Hold the swab above the breakpoint indication line and using sterile technique, collect the sample.
3. Unscrew and remove the cap from ESwab tube making sure not to spill the liquid transport medium.
4. Break the swab off into the tube.
 - a) Grasp the swab shaft at the very end with the thumb and first finger.
 - b) Lean the part of the shaft with the breaking point against the rim of the tube.
 - c) Bend the swab shaft at 180 degrees. If needed, gently rotate the swab shaft to complete the breakage.
 - d) Discard the broken handle.
5. Replace the cap on the tube and secure tightly.
6. Apply patient identification label to the outside of the tube and send to the laboratory.

ESWAB COLLECTION KIT COMPONENTS

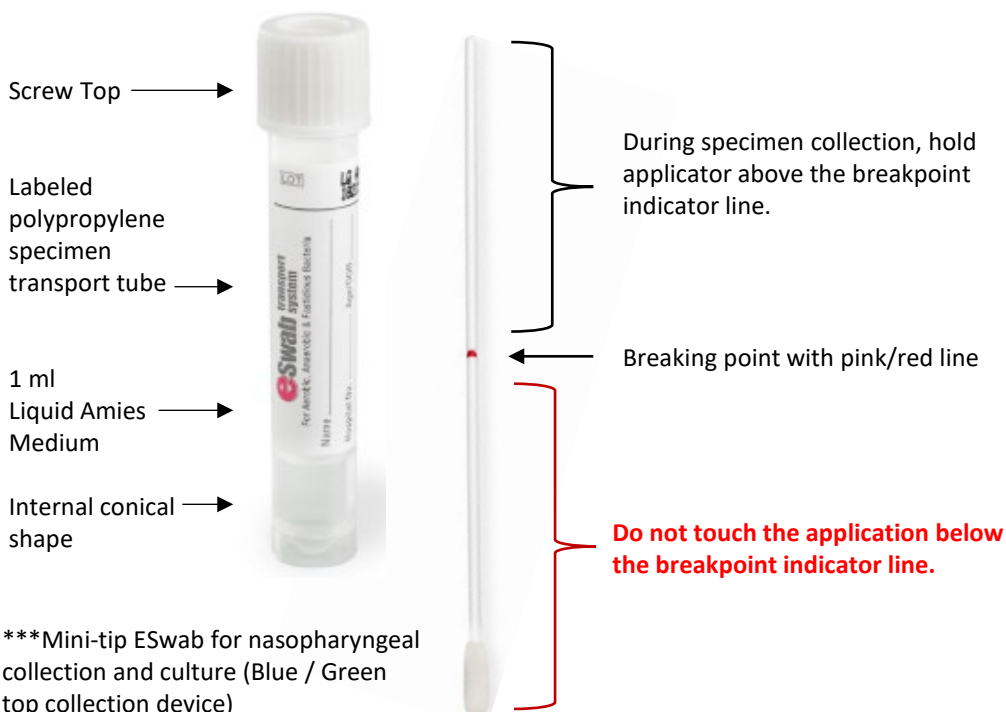

wisconsindiagnostic.com



Client Services (24/7):
414.805.7600

Specimen Processing
Wisconsin Diagnostic Laboratories
8777 W. Connell Ave.
Milwaukee, WI 53226

